

# Cavex GreenClean

Instructions for use: **Cavex GreenClean**

## 1. Introduction

**Cavex GreenClean** is a product for the removal of alginate or plaster residues from impression trays, spatulas and other instruments. **Cavex GreenClean** does not work aggressively but breaks down the structure of the alginate or plaster, thus making it easy to rinse or brush it off the tray or spatula. **Cavex GreenClean** is light green in colour and has a pleasant citrus aroma. The contents of the pack are sufficient to clean at least 500 impression trays.

## 2. Indications

Use **Cavex GreenClean** exclusively for removing alginate or plaster residues from impression trays and spatulas.

## 3. Use

- Dissolve 2 scoops (50 grams) of **Cavex GreenClean** in 0.5 litres of lukewarm water.
- Remove as much as possible of the alginate and plaster residue from the tray or spatula.
- Immerse the tray or spatula in the solution for 15 to 30 minutes.
- Remove the tray or spatula from the solution and brush away any residue that still remains. Rinse with water.

The use of warm water and/or an ultrasonic bath accelerates the effect of **Cavex GreenClean**.

## 4. Storage

The product has a shelf life of 3 years provided that it is kept in dry storage. Avoid direct sunlight.

## 5. Safety

**Cavex GreenClean** is pH neutral and does not contain any toxic components according to the Dangerous Substances Directive 67/584/EEC. **Cavex GreenClean** is biodegradable and can therefore be poured away directly after use without any pretreatment.

**General Conditions** Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.