CARIES DETECTION DEVICE

CARIES DETECTOR

CE
0197 93/42/EEC

GB INSTRUCTIONS FOR USE

Clinical procedure

Softened and markedly discoloured dentin must be removed from the carious element. A drop of the CARIES DETECTOR is placed in the cavity for approximately 10 seconds and then washed out with water. Only the outer layer of carious dentin is stained red. Completely remove the stained tissue, carefully sparing the unstained tissue. Enamel with residual caries and dental plaque will also be stained red and must be removed. Repeat the check with the CARIES DETECTOR until no more stained dentin shows. In no case, the CARIES DETECTOR should be used longer than 10 seconds, or otherwise the inner layer of carious dentin will also be stained.

Information

With the aid of this technique the dentist can be sure, that he has removed all infected carious dentin. The carious free dentin and all remineralizable dentin are thus preserved. Another advantage: CLEARFIL BONDING SYSTEMS are adhesive even to the inner layer of remineralizable dentin and can be placed immediately without undercuts. Finally this technique results in shallower cavities (because of preservation of the remineralizable dentin), so that possible pulp irritation will be reduced to a minimum.

Precautions

1. To avoid cross-contamination or injection, do not apply the product directly from the bottle to caries cavity.
2. If the product accidentally gets into the eye, flush immediately with copious amount of water and consult a physician.
3. If the product comes in contact with the skin or clothing, wash with soap and water immediately.
4. The product is flammable. Keep it away from open flame.

Storage precautions

1. Do not use after expiration date. Note expiration date on outside of package.
2. Do not store at temperature above 25°C. Avoid direct exposure to sunlight. Close container tightly after use and keep away from children.

CLEARFIL is a trade mark of KURARAY CO., LTD.

Warranty

KURARAY MEDICAL INC. will replace any product that is proved to be defective. KURARAY MEDICAL INC. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

Manufactured by
KURARAY MEDICAL INC.
1621 Sakazu, Kurashiki,
Okayama 710-0801, Japan

European Union Representative
KURARAY EUROPE GmbH
Building F821, Hoechst Industrial Park
65926 Frankfurt am Main, Germany
Phone:+49 (0)69 305 35 840 Fax:+49 (0)69 305 35 640

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