# TEETHMATE™ F-1

## LIGHT CURED PIT & FISSURE SEALANT

CE  
0197 93/42/EEC

### ENGLISH Instructions for use

#### I. INTRODUCTION

Teethmate F-1 is a light-cured pit and fissure sealant containing fluoride methacrylate-methyl methacrylate copolymer which provides fluorine.

#### II. INDICATIONS

Pit and fissure sealant

#### III. CONTRAINDICATION

Any histories of hypersensitivity to methacrylate-related monomers are contraindicated.

#### IV. PRECAUTIONS

##### <Safety Precautions>

##### For patients

- If any hypersensitivity occurs, discontinue using TEETHMATE F-1 immediately and consult a physician.
- If the etching agent or resin gets in the eye, immediately wash the eye with copious amounts of water and then consult a physician.
- If resin contacts the oral tissues, immediately wipe it off with a cotton pledget and then wash with copious amounts of water.
- If resin comes in contact with the skin, immediately wipe it off with a cotton pledget moistened with alcohol and then wash it with copious amounts of water.
- The etching agent is a 40% phosphoric acid. Avoid swallowing of the etchant. If the etching agent comes in contact with oral tissues, skin or clothing, immediately wash the area with copious amounts of water.

##### For practitioners

- Use protective gloves when using TEETHMATE F-1 to avoid any possibility of a hypersensitivity reaction.
- Discontinue using TEETHMATE F-1 when any hypersensitivity occurs, and then consult a physician.
- If the etching agent or resin gets in the eye, immediately wash the eye with copious amounts of water and consult a physician.
- If the etching agent or resin contacts the skin, immediately wipe it off with a cotton pledget moistened with alcohol, then wash the area with copious amounts of water.
- Keep the product out of reach of children.

##### <Handling and Manipulation Precautions>

##### General

Use a rubber dam to prevent moisture contamination. Moisture contamination may affect the adhesion effectiveness and properties.

##### Resin material

- The opaque shade contains small ball-bearings that are included to ensure a homogeneous mixture of the resin ingredients. Before each use, shake the container vigorously to assure mixture of any sediment.
- Replace the applicator cap or nozzle cap as soon as possible after use to prevent resin setting in the applicator.
- Use a polymerization activator equipped with a halogen lamp.
- Avoid unnecessary exposure to direct sunlight or operator lights when attaching the applicator nozzle.
- The resin material is in a gel-state when refrigerated. Bring it to room temperature before using.
- Change the applicator nozzle with each patient to avoid cross contamination.

##### Etching Agent

- Be careful that saliva or other exudates do not contact the etched area; this contamination would impair the adhesion effectiveness. If there is contamination, re-etch the tooth.
- Change the disposable brush tip for each patient to avoid cross contamination.

##### <Storage Precautions>

- Do not use after expiration date. Note expiration date on the outside of package.
- The product should be refrigerated when not in use (2°C-8°C); bring it to room temperature before using.
- Keep away from extreme heat or direct sunlight.

#### V. CLINICAL PROCEDURES

Read PRECAUTIONS carefully before using.

- Isolation:**  
Use a rubber dam to prevent moisture contamination.
- Cleaning of tooth surface:**  
Using a polishing brush cone in a slow-speed handpiece, clean the occlusal surfaces thoroughly while applying a constant stream of water.
- Cleaning the pits and fissures:**  
Using an explorer, thoroughly clean the pits and fissures.
- Acid etching:**  
Dispense K-ETCHANT GEL onto the mixing plate; then use the small disposable brush to apply the gel. Allow the etchant to remain in place for about 40 seconds; then flush with water and dry completely.
- Resin placement:**  
Remove the applicator cap and attach the applicator nozzle. Bend the tip for easy access and convenient handling. Place the applicator nozzle tip against the tooth pits and fissures and gently squeeze the barrel of the applicator for precise resin placement. If another application instrument, such as an explorer, is used, dispense the resin onto the mixing plate; then apply with the instrument.  
Avoid excess sealant placement and be sure to remove any excess resin with a cotton pledget prior to light-curing.  
Resin should not be exposed to opposing dentition. After placement of the resin, be sure to replace the applicator nozzle cap or remove the applicator nozzle and replace with the applicator cap.
- Light curing:**  
Hold the emitting tip of the curing light vertical to the resin surface and about one(1) mm away; then light-cure for 20 seconds.  
If the area to be sealed is wide, divide the surface into section of five(5) mm or less and expose each section for 20 seconds.
- Sealing evaluation:**  
Wipe away any unpolymerized resin with an alcohol moistened cotton pledget; then check the occlusal surface for completeness of coverage.

##### [WARRANTY]

KURARAY MEDICAL INC. will replace any product that is proved to be defective. KURARAY MEDICAL INC. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

##### [NOTE]

TEETHMATE is a trade mark of Kuraray Co.,Ltd.

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TEETHMATE is een handelsmerk van Kuraray Co. Ltd.
**KURARAY EUROPE GmbH**