



BY white 38%
X-TRA



BY white Powder & Liquid

whiten to B1 or
better in...
just 30 min

skySystem

“Bleaching Protocol”

1. Shades have to be assessed using a value ordered standard Vita shade guide. Record all shade measurements at baseline, and, if possible, take an initial photograph of the upper and lower teeth
2. Clean the teeth to be bleached by pumicing the facial surfaces. Check that the teeth to be treated are perfectly clean
3. Isolate with cotton rolls in the vestibule and lightly air-dry the teeth. Apply a cheek retractor
4. Protect the gingiva with **“BLEACH MASK LIGHT CURING”**. The gingival dam can be polymerized by using **“SKY LIGHT EASY”** (Time = 1 min.)
5. The patient has to be protected with the **“Orange Glasses”** included in the kit
6. Syringe the **“BY X-TRA 38%”** or **“BY WHITE POWDER&LIQUID”** in-office gel onto the teeth, being careful not to overfill.
7. Use **“SKY LIGHT EASY”** with time = 10 min. Push the light source close to the cheek retractor and press **“light on/off”** button. At the end of the whitening cycle the LED will turn off automatically. Remove the gel from the teeth. Do not wash with water.
8. Do again point n° 5 and n°6 for one time. Normally 2 whitening cycles are enough. In case, do again point n° 5 and n°6 for one time for optimal results.
9. Remove the gel from the teeth and the gingival dam from the gingiva. Wash with cool water. In case, apply **“BY SEALANT POTASSIUM NITRATE”** to avoid problems of sensibility.

BY
DENTAL

dental care products

Via Vecchia Prov.le Lucchese, 49/F
51030 Serravalle Pistoiese - PISTOIA - Italy
Tel: +390573994355 - Fax: +390573919480
www.bydental.com - info@bydental.com



Made in Italy