



sky light



- **DISPLAY:** working time. .
- **ON/OFF:** main power switch.
- **TIME ( Up and Down):** for setting the duration of the whitening cycle. The light source can be turned on for a minimum of 1 min. and a maximum of 15 min.
- **LIGHT On/Off:** to start the whitening cycle .Press "Pause" for momentarily interrupting the whitening cycle. Press again "Pause" to restart the whitening cycle. Press "Light On/Off" to reset the whitening cycle. A buzzer will advice when the whitening cycle is ending. Automatic stop at the ending of the whitening cycle.
- **Battery:** Red = it need to be charged  
Green Blinking = battery is charging  
Green = battery is charged

**BY**  
DENTAL

dental care products