ENGLISH

WARNING:

- For professional use only.
- Patient must wear protective eyewear.
- Operators must wear gloves, mask and protective eyewear.
- Do not use on pregnant or lactating women.
- Do not use on children under 14 years of age.
- Do not use the Gingival Barrier on any persons having known resin allergies.
- Do not use Pola Office on any persons having known peroxides allergies.
- Anyone with a history of chemical allergies is advised to undergo allergy checks before treatment.
- Do NOT anaesthetize patient.
- Pola Office Liquid syringe may be under pressure – remove cap carefully.
- Do not allow mixed gel or Pola Office Liquid to come in contact with skin, eyes and soft tissues.
- Pola Office will not lighten any restorative materials.

FIRST AID:

- SKIN/TISSUES (contact):** Immediately wash thoroughly with water. Apply a slurry of sodium bicarbonate and water to the affected area. Seek medical attention if symptoms persist.
- EYES (contact):** Open the eye wide and thoroughly wash for 15 minutes with running water. Seek medical attention.
- INGESTION:** Rinse mouth with water. Gargle with salt water and drink lots of milk. Seek medical attention if symptoms persist.
- INHALATION (powder):** Remove to fresh air. If breathing is difficult, seek medical attention.

TIPS:

- Patient must have good overall health.
- Seal all exposed roots and restorations.
- Inform patient that results are not guaranteed as the removal of stains varies between patients.
- Acid etching before treatment is NOT recommended.
- Large diameter suction tips are spread the gel to other areas.
- Mild discomfort in teeth during treatment can be a common side effect.
- If patient experiences sensitivity during light curing, move the tip away slightly. If this doesn't work, remove gel and rinse the affected area. Assess other possible causes.
- Delay any aesthetic restorations for 2 to 4 weeks after treatment.
- Advise patient not to smoke, consume highly coloured, hot or cold foods or beverages for at least 2 days.
- Treat any post-operative sensitivity with a desensitising gel or toothpaste.
- Prescribe Pola Day or Pola Night to maintain brightness if desired.
- MSDS available at www.sdi.com.au
- A patient consent form is available for download at www.sdi.com.au

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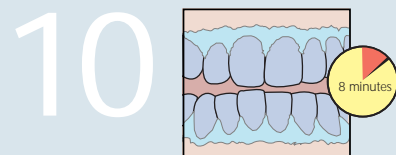
Leave gel on for 8 minutes (Optional: a curing light may be used at this stage, please refer to light manufacturer's recommendations)

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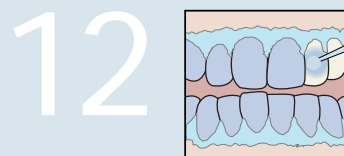
Complete Steps 9-11 three times (optional 4 times).
Note: One pot should only be used for 2 applications or within 20 minutes of mixing.

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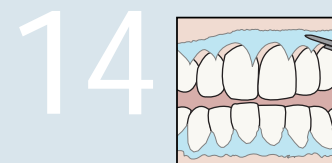
Remove Gingival Barrier by lifting it from one end.

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ENGLISH

Instructions for non-vital tooth whitening.

1. Verify adequate endodontic obturation.
 2. Isolate tooth with rubber dam.
 3. Open lingual access to pulp chamber and remove all composite base material and gutta percha 2-3mm apical to the cemento - enamel junction.
 4. Place glass ionomer base 1mm thick to ensure a barrier between the "sealed" root canal and whitening gel.
 5. Place and light cure a thick layer of Gingival Barrier at the gingiva region of the isolated tooth.
 6. Mix the Pola Office gel as instructed in steps 6-8 of the vital tooth bleaching instructions.
 7. Place the gel into the opened pulp chamber and onto the labial surface.
 8. Leave gel on for 8 minutes.
(Optional: a curing light may be used at this stage, please refer to light manufacturer's recommendations).
 9. Suction off using a surgical aspirator tip.
 10. Complete steps 7-9 until the desired shade is reached. Note: One pot should only be used for 2 applications or within 20 minutes of mixing.
 11. Wash and apply suction after the last application.
 12. Remove Gingival Barrier and re-seal tooth with appropriate restorative.
- Note:
Recall patients periodically for radiographic and clinical examination.

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Pola Office

- 35% hydrogen peroxide
- in-office whitening system
- up to 9 shades lighter
- total treatment time only 30 minutes
- contains desensitizers

Pola Office 3 Patient Kit

Reorder 7700015
6 x 0.3g Pola Office powder pots
6 X 2mL Pola Office syringes
3 x 1g gingival barrier syringes
accessories

Pola Office 1 Patient Kit

Reorder 7700030
2 x 0.3g Pola Office powder pots
2 X 2mL Pola Office syringes
1 x 1g gingival barrier syringe
accessories

COMPOSITION

• Pola Office Liquid	Hydrogen peroxide
35%	Water
64.95%	Stabilizers
<0.05%	
• Pola Office Powder	Thickeners
73.2%	Catalysts
26.2%	Dye
<0.04 %	Desensitizing agents
0.5%	
• Gingival Barrier	Methacrylic ester
83.95%	Silica
16%	Pigment
<0.04%	Butylated hydroxy toluene
<0.01%	

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Made in Australia by SDI Limited
Bayswater, Victoria 3153
Australia 1 800 337 003
Austria 00800 022 55734
Brazil 0800 770 1735
France 00800 022 55 734
Germany 0800 100 5759
Ireland 01 886 9577
Italy 800 780625
Japan 0120 98 2770
New Zealand 0800 734 034
Spain 00800 022 55 734
United Kingdom 00800 022 55 734
USA & Canada 1 800 228 5166
European Union Representative:
SDI Dental Limited
Block 8, St Johns Court
Swords Road, Santry
Dublin 9, Ireland
Tel +353 1 886 9577
www.sdi.com.au

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* Vita shade guide is a trademark of VITA Zahnfabrik, H. Rauter GmbH & Co. KG, D-79713 Säckingen, Germany
* Die Vita Skala ist ein geschütztes Produkt der Vita Zahnfabrik Rauter GmbH & Co. KG, D-79713 Säckingen
* O guia de cores Vita é marca registrada de VITA Zahnfabrik, H. Rauter GmbH & Co KG D-79713 Säckingen, Alemanha.
* Guia de tonos Vita es marca registrada de VITA Zahnfabrik, H. Rauter GmbH & Co. KG, D-79713 Säckingen, Alemania.
* Le nuancier Vita est une marque déposée de VITA Zahnfabrik, H.Rauter GmbH & Co. KG, D-79113 Säckingen, Allemagne.
* La scala colori Vita è un marchio registrato VITA Zahnfabrik, H. Rauter GmbH & Co. KG, D-79713 Säckingen, Germania.