



COMPANY

PRODUCTS

NEWS

MEDIA


DISTRIBUTORS

pola office+

Super Fast

Minimal chair time is required. The total treatment will take less than 30 minutes. Less chair time means increased patient comfort and satisfaction.

The unique 37.5% hydrogen peroxide formula ensures fast release of the peroxide ions to begin the bleaching process.

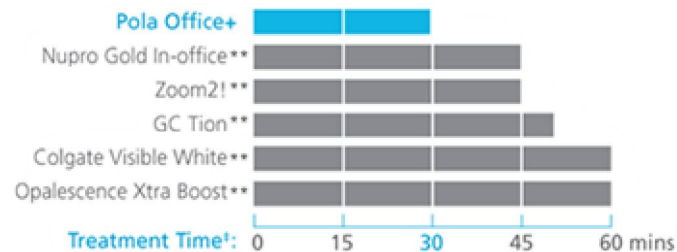

No bleaching light required

Pola Office+ can be used with or without a bleaching light, as it does not require activation. While a light is not necessary, you certainly can use a bleaching light or a standard heat emitting curing light to accelerate the bleaching process and

Simple delivery

No mixing is required. The dual barrel syringe system mixes as you apply the gel directly to the tooth. This ensures a consistent blend of freshly activated gel. The fine applicator tip allows easy application, especially in tight spots.

impress your patients.

Maximum comfort

The neutral pH of Pola Office+ ensures maximum comfort during and after treatment. The shorter treatment time results in less patient discomfort and lower sensitivity.

Built-in desensitizer

No separate desensitizer is required. Pola Office+ has unique built-in desensitizing properties that inhibit post-operative sensitivity. Potassium nitrate, the key pain-reducing ingredient, penetrates the nerve endings by blocking transmission of sensitive nervous impulses to provide a calming effect.


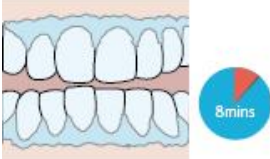

Just one thin layer


A thin layer of gel is all that is required. Use the tip to spread the gel ensuring all tooth surfaces are covered. The sticky gel formula adheres firmly to tooth surfaces maximizing the bleaching effect.

"89% of evaluators
would recommend
Pola Office+
to a colleague¹"

"83% of evaluators
would purchase
Pola Office+¹"

Instructions

- 1 Record pre-operative shade.
- 2 Clean teeth with flour based pumice.
- 3 Place cheek retractor and cover exposed lip with petroleum gel.
- 4 Dry teeth and apply gingival barrier; light cure in fanning motion.
 
- 5 Firmly attach a mixing tip to Pola Office+ syringe away from patient; dispense small amount of gel onto mixing pad to ensure adequate mixing.
- 6 Directly apply a thin layer of gel to all teeth undergoing treatment.
- 7 Leave gel on teeth surfaces for 8 minutes. Optional curing light can be used according to manufacturer's instructions.
 
- 8 Suction off using a surgical aspirator tip.
 
- 9 Complete steps 6-8 three times (optional 4 times).
- 10 After the last application, suction all gel off, then wash with water and apply suction.
- 11 Remove gingival barrier.

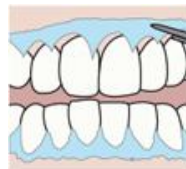

** Nupro Gold In-office, Zoom2!, GC Tion, Colgate Visible White and Opalescence Xtra Boost are not the trademarks of SDI

‡ Published and manufacturers data

¹ In-house data based on external clinical evaluators

For further information on [pola_office+](#)

• [Re-Order Codes](#)



LANGUAGE: · SEARCH:

Please direct all inquiries to [SDI Limited](#) | Report any problems to the [Webmaster](#) | [Privacy policy](#) | © 2009 SDI All rights reserved.