

Hemocollagene

Individually packed sterile sponges for dental use

Composition

Per sponge

Native non-denatured lyophilized collagen 0.027 g

Indication

Hemostasis after tooth extraction.

Directions for use

Before placing a Hémocollagène sponge, clean the socket thoroughly so as to remove all debris and especially tooth fragments.

Dry the wound and then apply the dry Hémocollagène sponge, with or without compression, depending on the degree of bleeding.

If necessary, the Hémocollagène sponge can be held in place by horizontal cross-stitching over the socket. Complete resorption will be achieved.

Warnings and precautions for use

- Do not use the Hémocollagène sponge in infected areas
- Do not expose the Hémocollagène sponge to moisture
- Hémocollagène should not be autoclaved
- Check that the sterility protection is intact before use
- Any non-used portion of sponge must be discarded since it will no longer be sterile.

It is advisable:

- To use smooth instruments
- Not to apply it in excess
- It is preferable to apply the drug directly upon the lesion, taking care not to moisten it beforehand (blood, saliva....).

Storage

In order to keep the dressing sterile and to prevent it from being remoistened, the thermosealed strip must be maintained in its integrity and kept in its bag until use.

Presentation

Box containing 24 sponges for single use.

Radiosterilized sponges under 25 laGy.

For professional use only in the practice of dentistry.